



VIVACE™

Facial Guidelines: Representative Image of Facial Dermal + Epidermal

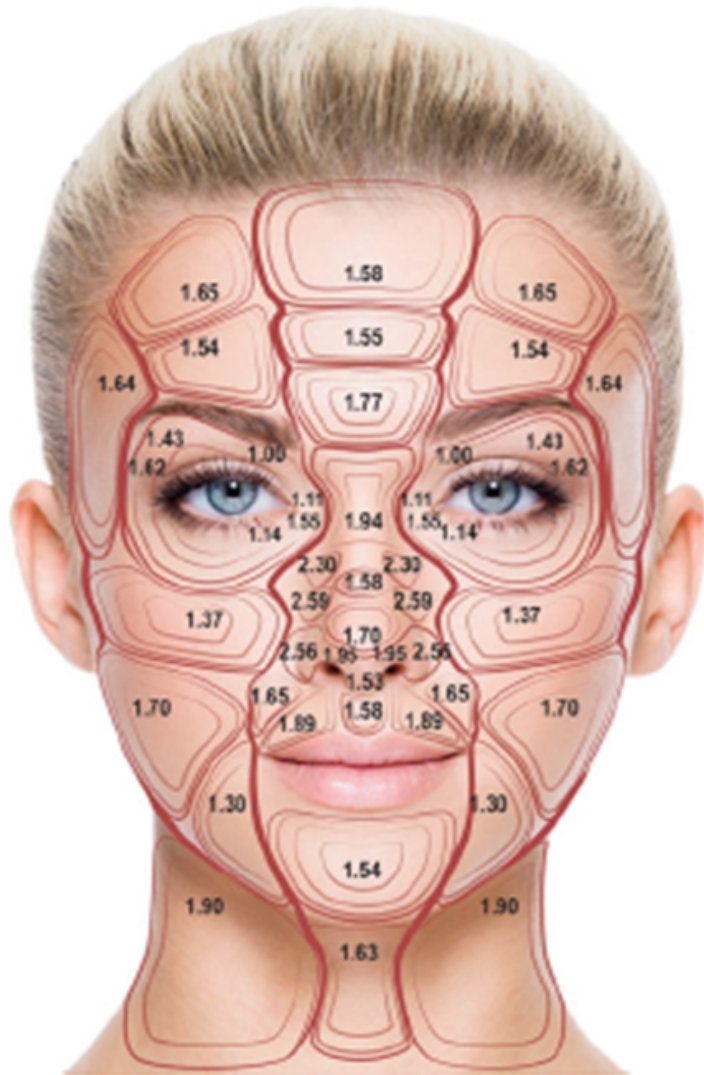


Figure 1. Anterior view of epidermal relative thickness (RT) values. Thickness in millimeters (mm).

Vivace™ Use: Ten (10) Key Considerations

1. Use adequate numbing topical anesthetic before Vivace™. ABM recommends a Benzocaine/ Lidocaine/ Tetracaine combination (BLT 20/10/10, applied at least 20 minutes prior to treatment).
2. Each patient is different with different pain tolerances/skin and therefore adjust power levels and RF time for individualized comfort.
3. When determining depth, use the facial guidelines found in treatment training guidelines sheet and use deepest setting first decrease depth by -0.2 to -0.5mm for 2-3 passes.
4. For best results, overlap 50% in each direction for each pass.
5. Be cautious about sensitive areas that are prone to patient discomfort.
6. If LEVEL is found to be too painful for the patient, try decreasing power level, while keeping RF-time the same.
7. Using the Boost Glide Serum and Soothe Recovery Mask post procedure significantly reduces inflammation and minimizes downtime (if any occurs).
8. If you are a provider with the device, please contact Device Hotline: devicehotline@aestheticsbiomed.com
9. If you are a provider with clinical questions, please contact Clinical Hotline: clinicalhotline@aestheticsbiomed.com
10. If you are a provider with other questions, please contact Customer Hotline: customersupport@aestheticsbiomed.com